



Instructions



SARS-COV-2 Neutralizing Antibody

Quantifying the amount of COVID-19 neutralizing antibodies your body is producing to fight off the SARS-CoV Virus

STEP 1: MATERIALS CHECKLIST

Blood Collection Supplies



x1

BD Microtainer SST™



x2

Alcohol Pad



x 2

Lancet



x 2

Band-aids



Gauze Pad


Shipping Supplies



Original Mailer Box



Biohazard Specimen Bag

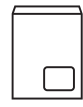


Mailing Envelope

STEP 2: FILL OUT INCLUDED REGISTRATION CARD

STEP 3: BLOOD SAMPLE COLLECTION

1

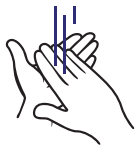


Wash hands in very warm water. Vigorously shake hands. Massage selected hand. Vigorously rub the selected finger stick area. Let your hand hang at your side and shake it back and forth for 15-20 seconds.

2



Select finger: middle or ring finger is preferable. Identify the desired puncture site and wipe it with alcohol wipe then wait for 30 seconds.

3

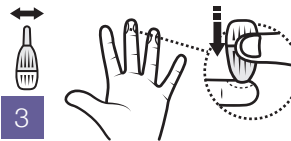

Twist and pull the blue lancet cap straight out. Position the safety lancet against the puncture site as illustrated. To activate, press the safety lancet firmly against the puncture site. Do not remove the device from the site until an audible click is heard.

4




Gently apply intermittent pressure along finger to the puncture site. Keep hand below heart level to promote better blood flow. Strong repetitive pressure (milking) must be avoided.

5



Wipe away the first drop of blood with a clean dry gauze pad.

6

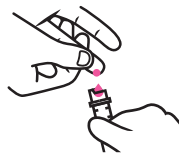

Position the BD Microtainer™ directly beneath the puncture site. Apply gentle pressure along finger to help ensure continuous blood flow. Ensure that the tube is not scraped over the puncture site. Fill the tube to the required blood level indicated on the tube.

7



Close the tube by twisting the cap back on and gently invert the Microtainer™ 5 to 10 times.

8



Clean your finger and apply a bandage.

STEP 4: PACK SAMPLES

Clearly write your name and date of birth directly on the tube in the space provided.

Make sure the tube is tightly closed and place it inside the biohazard specimen bag and seal the bag.

Place the biohazard specimen bag inside the original mailer box and close.

Place the original mailer box inside the mailing envelope, then ensure the return label is adhered to the mailing envelope.

STEP 5: SHIPPING SAMPLE

Simply drop in your mailbox or take to your nearest USPS.

IF YOU NEED HELP CALL (800) 820-8803

STEP 6: RESULT

Within **72 HOURS** after receiving your specimen, a link will be sent to your registered email or phone. Use the link to check your result.



SARS-COV-2 Neutralizing Antibody



Need help? We are here.

We're available by email or phone to answer your questions,
review your sample, and more.



(800) 820-8803



cs@ihdlab.com



Register



Sample Collection



Lab



Results

Scan for video
explainer instructions

